


Double Loop Bed Hand (135/135CP)

User Instructions

The double loop bed hand is constructed from tubular steel and fits between the bed base and mattress offering a rigid handle to aid the user(s) getting in and out of bed. It is suitable for beds between 3ft (single) and 4ft 6in (double) wide.

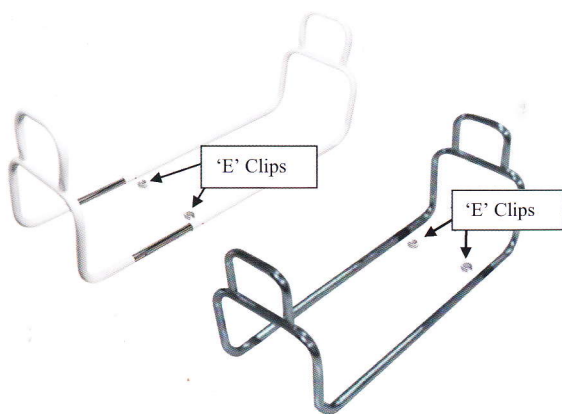

Installation Instructions

1. Remove the mattress from the bed, and place the Bed Hand on the bed base.
2. Remove the 'E' clips (arrowed above), and adjust the Bed Hand width, so that its internal width is the same width as the mattress and the holes in the tubing line up.
3. Reinsert the 'E' clips in to the holes to secure the Bed Hand at the correct width.
4. Place the mattress on top of the bed base and Bed Hand.

Operating Instructions and Safety Precautions

1. Use the loop on either side of the Bed Hand to assist with getting in and out of bed.
2. The device is suitable for users up to 140kg (22 stone). Do not exceed this weight limit.
3. The frame must be fitted in a level position between the bed base and the mattress.
4. When fitting a Double Loop Bed Hand a risk assessment should be carried out as to the suitability of the equipment for the user. Particular consideration should be given to the risk of entrapment.
5. The Bed Hand may not be suitable for all bed bases and/or mattresses. The Bed Hand is not suitable for use with electrically adjustable beds.
6. The device should be checked regularly to maintain the correct position for the user. This would have been determined following the risk assessment.
7. The device should be wiped down regularly with either a cream cleaner or a mild detergent.

Specification

Dimensions (cm):	44H x 34D x 96 – 156W
Dimensions (in):	17½”H x 13½”D x 38 - 61½”W
Weight limit:	140kg (22 stone)
Product weight:	4kg (9lb)

Warranty

The device comes with a 12-month warranty from date of purchase, to cover defects in design or manufacturing. It does not cover accidental damage or wear and tear. Full warranty conditions are available from your retailer.