

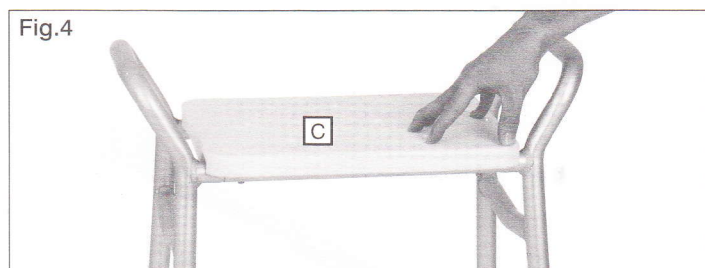
Homecraft Lightweight Adjustable Height Shower Stools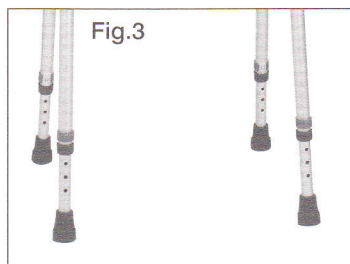

	Description	Old Code	New Code
1	Fixed	539L	09 132 5356
2	Flat Packed		09 156 4749
3	Optional Seat Cover		09 156 4756

Please read these instructions carefully and retain for future reference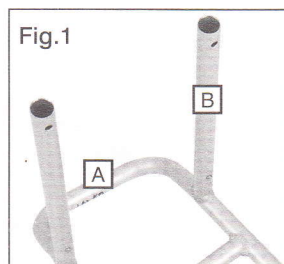

Maximum user weight

Introduction

This Homecraft Shower Stool is a lightweight, height adjustable, non-rusting shower stool manufactured from aluminium tubing and fitted with a plastic seat. It has built-in side handles which offer stability and additional support to the user. There is an optional seat pad for additional comfort and luxury (sold separately).

Intended Uses

To compensate for a disability by providing a sitting surface for a person taking a shower.

Assembly Instructions (Flat packed version only)

1. Remove all packaging
2. Take the A-shaped frame (A) and attach the cross bars (B) ensuring the push pins are fully protruded, see **fig.1**
3. Repeat for the other side of the shower stool, see **fig.2**
4. Slide the legs into the base of the frame ensuring that the holes in each section are aligned
5. Clip in the 'snap locks' at the required height, ensuring all four legs are positioned at the same height, see **fig.3**
6. Set the plastic seat (C) over the cross bars and push until the seat clips into position, see **fig.4**
7. A correctly fitted shower stool is featured in **fig.5**

Height Adjustment

8. To adjust the height grasp the 'snap lock' between the thumb and forefinger and pull it out
9. Slide the leg in or out to the required position
10. Push the clip back firmly into the hole
11. Repeat the same procedure for the other legs making sure all four legs are adjusted to the same height

Care and Cleaning

The Homecraft Aluminium Shower Stool is designed for minimum maintenance. To clean, wipe over with a wet soft cloth. Use general purpose neutral detergent and warm water. The shower chair may be placed under running water at low pressure or showered down to assist cleaning. The plastic seat can be unclipped to make cleaning easier. Do not use harsh abrasives or bleach based cleaners to avoid discolouration. To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (Haztabs/Precept), rinse and dry.

Warranty Information

The Homecraft Shower Stools are warranted for one year from date of purchase due to faulty workmanship and materials. Please contact your supplier/dealer should a fault occur. The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses etc. The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.

CAUTIONS

- Under no circumstances should the shower stool be used for standing
- Ensure users feet are firmly placed on the floor when using the stool
- Ensure all four legs are positioned at the same height
- Never exceed the stated maximum user weight of 190kg / 30st
- Do not store in sub-zero temperature
- Periodically check for elongated or stressed adjustment holes
- Check for split, worn or loose rubber ferrules and Snap Locks
- Do not use with a damaged or split seat
- As there is no back support, do not lean back when using this shower stool

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Performance Health International Ltd.
Performance Health Holdings, Inc.