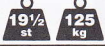
Code	Gebruikersgewicht maximaal
09 154 8494	 19 1/2 st 125 kg

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- A** Bell
- B** Backrest
- C** Handle Height Adjustment
- D** Shopping Basket
- E** Crutch Holder
- F** Folding Bar
- G** Kerb Climber
A kerb climber also makes manoeuvrability over thresholds easier
- H** Backrest lock
- I** Aluminium Frame
- J** Basket Support Tube
- K** Easy to use backrest adjustment
- L** Simple folding function
- M** TPR (synthetic rubber) wheels for increased stability and grip

Assembly Instructions

- Remove all contents from the carton
- Push the folding bar **F** to lock the Rollator in the open position
- Insert a handle with the pre-assembled locking loop brake into each handle tube. Ensure the brake cable runs down the outside of the frame.
- Adjust each handle to the desired height, lining up the holes appropriately. Slide the bolts through the holes from the inside of the frame and screw on the tightening knob until secure. Ensure the bolt heads are properly countersunk within the holes.
- Fold up the pushing handle **A** and insert into frame receiver bracket **H**
- Attach the basket to the front of the using the vertical hooks
- Remove the protective plastic covering from the tyres.

Folding the Rollator

- To unfold the walker push the handle on the front of the seat down until the seat is fully folded down and locked. Make sure the walker is fully extended (sitting down) until the clips on the underside of the seat are attached to the walker tube.
- To fold the walker lift the handle at the front of the seat until the walker is fully folded.

Hand Brake Operation

Note: Always engage and disengage brakes on both sides.

- To apply the brakes, squeeze both brake handles. This can be used to slow or stop the Rollator.

To lock the brakes:

- Push your palms down on the hand brake. When the wheels lock you will feel and hear a "click". The brakes are now locked.

To unlock the brakes:

- Pull up on the brakes with your palms. On release, you will feel and you will hear a "click". The brakes are now unlocked.

Safety Precautions

Read Before Use

- Ensure the Rollator is fully open, the seat is down and clips on the underside of the seat are mounted into the walker frames to secure the Rollator in the correct position.
- Ensure both handles are at the same height and that the tightening knobs are fully tightened. Test the Rollator for stability before applying full weight to the handles.
- Test the brake mechanism. When the wheels are locked, the brake pad should prevent the wheels from turning. If the wheels turn while the brakes are locked, refer to point 4 of the Care and Maintenance section for adjustment instructions. When the brakes are released, the wheels should turn freely.
- Check that the Rollator is free from damage and excess wear by carrying out the checks detailed in the Care & Maintenance section.
- Do not leave the Rollator unattended without locking the brakes.
- Do not use the Rollator as a seat or in any type of motor vehicle.
- Always lock the brakes before sitting.
- Do not stop, turn or reverse on a ramp or incline.
- Do not make sharp turns.
- Do not use the seat if the walker on a slope or on uneven ground.

The brakes must be in the locked position (parking) before using the seat

Care & Maintenance:

- The rollator should be inspected before each use to ensure it is free from damage and excessive wear.

Check that:

- The handgrips are not loose or damaged.
 - The brake cables are not damaged in any way.
 - The wheels are securely in place and are not damaged or excessively worn.
 - The frame is not cracked or dented.
 - The frame is secure, stable, and all nuts and bolts are tight.
-

If you do find any wear or damage to your rollator, DO NOT USE.

- Check that all wheels and folding parts move freely.
- Keep the rollator clean by wiping with a damp cloth on a regular basis.

Do not use abrasive cleaning materials

Specifications

- Name: GoGo Light Aluminium Rollator
- Maximum User Weight: 125kg / 19½st
- Handle height: 810-925mm with 7 settings
- Frame Width: 485mm
- Backrest Height: 810mm
- Seat Height: 565mm
- Overall size: 610×680×810mm (min)
- Front wheel to Rear wheel: 490mm
- Total Weight: 8.4kg
- Accessories: Storage Basket, Crutch Holder, Kerb Climber, Bell

Warranty

Patterson Medical Ltd warrants that the rollator is free from defects in material and workmanship. This warranty shall remain in effect for one year from the date of original consumer purchase.

This warranty does not cover the following:

- Replacement parts supplied by anyone other than Patterson Medical Ltd or the distributor.
- Any failure of the unit during the warranty period if the failure is not caused by defect in material or workmanship or if the failure is caused by unreasonable use.

Patterson Medical Ltd is not liable for incidental or consequential injury or damage to property.

Should it be necessary to return the rollator, it is essential it is adequately packed to protect it in transit, preferably in its original box; otherwise Patterson Medical Ltd cannot accept any responsibility for transit damage to the unit.

This does not affect your statutory rights.

Manufactured for Patterson Medical Ltd.

Nunn Brook Road, Huthwaite,
Sutton-in-Ashfield, Nottinghamshire,
NG17 2HU, UK.

UK: Tel: 08444 124 330 Fax: 08448 730 1000
www.pattersonmedical.co.uk

International: Tel: +44 1623 448 706
Fax: +44 1623 448 784

Patterson Medical Products, Inc.

28100 Torch Parkway Suite 700,
Warrenville, IL 60555-3938.
Tel: 1-800-323-5547 Telefax: 1-800-547-4333

Patterson Medical Australia

Unit 3, 3 Basalt Road, Pemulwuy
NSW, Australia 2145.
Tel: 1300 473 422 Fax: 1300 766 473

Patterson Medical France

Zone Industrielle de Montjoly,
11 Avenue de l'Industrie - CS 40712
08013 Charleville-Mézières Cedex, France.
Tel: 03 24 52 61 15 Fax: 03 24 52 90 34