TABLE DE LIT

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BED TABLE

ASSEMBLY INSTRUCTION

Check that all the parts shown on the diagram below are present.

- Assemble each part as shown on the diagram.
1. Insert the threaded rod (3) in the tray support (4), lining up the bevels in order to be able to insert the assembly in the tray support (2) which is to be butted up against the column.
 2. Place the stop washer and the knob (5) on the end of the threaded rod. Screw on moderately tightly.
 3. Screw the feet (8) to the base (7) making sure that the slot for the feet is correctly lined up with the base tube.
 4. Insert the support (4) in the base and then the screw (6) in the base (7).
 5. Adjust to the required height and tighten the screw moderately.
 6. Adjust the angle of the tray and tighten the knob (5).
 7. Check that all the castors touch the ground and that the product is stable when you use it.

INSTRUCTION FOR USE.

Read these instructions carefully before using the product. If in any doubt as to the use or the appropriateness of the product to your needs, contact your salesman or the person who prescribed the product.

1. Do not exceed the maximum weight of the user or the maximum load indicated on the product.
2. Assemble the product, if required, in accordance with the attached assembly instructions.
3. Before using, verify the product is stable and rigid, and that all the screws are properly tightened.
4. Do not use if the product is damaged. Contact your salesman to have the worn or faulty parts replaced.
5. Do not store the product at extreme temperatures (<0° - >40°) or in the open air.
6. Clean the product regularly with non abrasive domestic cleaners in accordance with their conditions of use. Dry with a soft cloth.
7. To disinfect the product where it is being used by several people (in an institution, for example) follow the recommendations of the person responsible for hygiene.

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