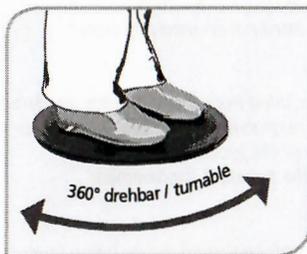
Technische Daten / Technical specifications:

Art.-Nr. / Item No.: 46 800 Drehscheibe / Turntable			
Material:	Kunststoff / Plastic	Durchmesser / Diameter:	ca. / approx. 40 cm
Farbe / Colour:	schwarz / black	Gewicht / Weight:	1 kg
Höhe / Height:	ca. / approx. 2 cm	max. Belastung / max. Load:	125 kg
Lebensdauer / Lifespan:	ca. 10 Jahre / approx. 10 years		

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GB**Usage:**

The turntable facilitates to turn over a patient who is still able to sit or to take over some weight with the feet (for example, from the bed edge into the toilet chair.) Due to the reduced effort, the carer and the patient are considerably relieved.

Indication:

For patients with movement and functional disorders in the area of rotation of the body around the longitudinal axis. (The turntable can be rotated by 360° in both directions.)

Before each use:

*Please check before each use, if the turntable is damaged (**46800:** if all 6 anti-slides are present.) If anti-slip knobs are missing, the turntable must not be used.*

Use:

*The turntable is placed down on the floor (**46800:** with the anti-slip knobs.) The patient's feet are placed in the middle of the turntable and the carer has to put a foot on the tool before the patient is raised. This avoids unwanted rotation of the turntable. When the patient has been raised, he can be rotated with the rotary disc into the desired position and lowered there again.*

Maintenance:

The turntable does not require any special maintenance, but has to be regularly checked for cracks or damage.

Cleaning / Disinfection:

Do not use sharp or abrasive objects for cleaning, please use commercially available cleaning detergents.

Disinfection: All commercially available disinfectants suitable for plastics can be used.

Important: Please make sure to observe the manufacturer's instructions.

⚠ Safety instructions:

- ▶ *The body weight of the user must not exceed 100 kg (46800) or 125 kg (46810)!*
- ▶ *The turntable must never be used alone, but must be used with a caregiver (care worker)!*
- ▶ *Always use the center of the turntable.*
- ▶ *Do not allow the anti-slip knobs to come into contact with oily or soapy substances, slippery hazard!*
- ▶ *Use only level and dry floors, risk of skidding!*
- ▶ *Do not use the turntable in the wet area (bathroom) or on carpet bridges.*
- ▶ *The turntable is not a toy, do not leave children unattended with the product!*
- ▶ *The turntable can be used in the temperature range of 5 - 70 °C.*
- ▶ *If parts of the turntable are damaged, they must not be used any more!*

Warranty:

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly. We reserve the right of modifications serving the technical improvement.

Note:

Modifications or repairs of the turntable must only be carried out by trained personnel using original spare parts.