Marks	Designations	A	B	C	D	E	F	G	H	Weight	
3005**	Open Standard seatplate	86 cm	51.5 cm	60.5 cm	72 cm	52 cm	38 cm	39 cm	58 cm	6.6 kg	
3003**	Open Full seatplate				52 cm						
3015**	Open Adjustable with standard seatplate	86/97 cm	51.5 cm	60.5 cm	72/83 cm	52/63 cm	38 cm	39 cm	58 cm	6.8 kg	130 kg
3014**	Open Adjustable with full seatplate				72/83 cm						
3330**	Club with standard seatplate	91 cm	42 cm	54 cm	67 cm	45 cm	50 cm	44 cm	53 cm	8 kg	130 kg
3300**	Club with full seatplate				67 cm						
3400**	Confort with standard seatplate	91 cm	42 cm	54 cm	67 cm	45 cm	50 cm	44 cm	53 cm	9 kg	130 kg
3100**	Confort with full seatplate				67 cm						
3030**	Even with standard seatplate	87.5 cm	51.5 cm	64.50 cm	75 cm	51 cm	38.5 cm	40 cm	42 cm	7.5 kg	130 kg
3031**	Even with full seatplate				75 cm						
3200**	New club with standard seatplate	84 cm	53 cm	61 cm	73 cm	52 cm	35 cm	40 cm	49 cm	7 kg	130 kg
3201**	New club with full seatplate				73 cm						
3001**/ 3002**/ 3003**	Club XXL with standard seatplate	92 cm	55/65/75 cm	61/71/81 cm	67 cm	46.5 cm	46 cm	44 cm	53 cm	10 kg	160 kg
3101**/ 3102**/ 3103**	Confort XXL with standard seatplate				67 cm					10.5 kg	
3061**/ 3062**/ 3063**	Club XXL with full seatplate	92 cm	55/65/75 cm	61/71/81 cm	67 cm	46.5 cm	46 cm	44 cm	53 cm	10 kg	160 kg
3161**/ 3162**/ 3163**	Confort XXL with full seatplate				67 cm					10.5 kg	
3046**	Best Up	94.5cm	44cm	47cm	54 cm	48.5cm	50cm	39.5 cm	45 cm	9.4kg	130kg
3646**	Best Up XXL	88 cm	75 cm	87 cm	75 cm	56 cm	59 cm	44 cm	59 cm	9.5kg	160 kg

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Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Before seating make sure that you have correctly placed the seatplate on the chair.
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips)..
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a commode chair and in proper way : don't stand up on the chair.
- ♥ ALWAYS observe the weight limit on the labelling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the chair, make sure that it is securely locked in OPEN position and level to the ground before using. Some chairs are already mounted.
- ♥ DO NOT hang anything on the Chair.
- ♥ ALWAYS test to see that the chair and attachments are properly and securely locked in place before using.
- ♥ In case you didn't use your chair for a long time, let it checked by the technical staff, before using.
- ♥ Do not store outdoor, the chair must be used at home.
- ♥ Do not install or remove the pan while the patient is seating on the chair! High risk of sex injuries.
- ♥ Be carefull when removing the armrests to avoid trapping your fingers.
- ♥ Check the chair before using, in case of doubt ask your physician or provider.
- ♥ Use only the chair on a flat and solid ground.
- ♥ It is forbidden to seat on the armrests.
- ♥ Do not lean out of the chair to avoid losing balance.

2. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

Mounting the bucket

There are 2 different ways to install/remove the bucket :

- 1- from the left or right side under the seat, insert the pan into the metallic support (single possibility for XXL models).
- 2- lift up the seatplate and remove the bucket from the top, see photo [B]. Close the bucket with the lid before transporting it.

Mounting the seatplate

The seatplate can be easily removed for cleaning or for replacement as it is only connected to the frame with 2 clips see photo [A].

To remove the seatplate, take out the clips from the frame by lifting them up, see photo [B].

To mount the seatplate, clip the two clips on the back pipe of the metal frame, see photo [C].

By adding the cover on the hole, the commode chair can be used as a regular chair.

Rental: this chair can be rented under the condition that it was correctly cleaned. If necessary, replace the lid and the seatplate before the next rental. This user's manual belongs to that product and must be delivered to each new user.

3. Cleaning

The chair has to be cleaned regularly and after each use with a mild cleaning agent and a soft cloth.

4. Materials

The product is made up of steel tubing covered with epoxy painting. Upholsteries are in PVC. The chair is delivered with a pan. Some models are equipped with swinging armrests.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

See tabel page 1

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible.

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