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Read these instructions carefully before using the product. If in any doubt as to the use or the appropriateness of the product to your needs, contact your dealer or the person who prescribed the product.

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1. Do not exceed the maximum weight of the user or the maximum load indicated on the product.
2. Assemble the product, if required, in accordance with the attached assembly instructions.
3. Before using, verify that the product is stable and rigid, and that all the screws are properly tightened.
4. Do not use if the product is damaged or worn. Contact your dealer to have the worn or faulty parts replaced.
5. Do not store the product at extreme temperatures (<0° -> 40°) or in the open air.
6. Clean the product regularly with non-abrasive domestic cleaners, in accordance with their conditions of use. Dry with a soft cloth.
7. To disinfect the product where it is being used by several people (in an institution, for example) follow the recommendations of the person responsible for hygiene.

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PRODUCTS ADJUSTABLE IN HEIGHT
 Contact your dealer or the person who prescribed the product so that he can adjust the product to the correct height. Check that all the feet have been set to the same height.

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1. Adjustment by clip : Ensure that the clip surrounds the tube properly and that it goes right through the tube.
2. Adjustment by push-button : Ensure that the push button is fully out of the tube.
3. Adjustment by black tightening handle : Ensure that the height adjustable handle cannot pivot.

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WALKING AIDS
 Only to be used on flat, firm surfaces. Replace worn ends or wheels. Adapt your speed of walking to your surroundings. Slow down on damp, cold or slippery surfaces. Hold on to the banister with one hand when going downstairs with walking sticks.

WALKING AIDS
 Only to be used on flat, firm surfaces. Replace worn ends or wheels. Adapt your speed of walking to your surroundings. Slow down on damp, cold or slippery surfaces. Hold on to the banister with one hand when going downstairs with walking sticks.

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HYGIENE AIDS
 Beware of slippery surfaces. Help should be given to users with restricted physical capacities.

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