W1840 004 201  
W1840 004 301  
W1840 005 201  
W1840 005 301

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**GB Installation instructions**  
Attach the 2 x L-shaped brackets to the raised toilet seat: turn the model upside down on a flat surface and insert the L-shaped brackets into the slots. Push gently on the bracket and turn the knob clockwise until the thread engages. Position the brackets leaving a greater width than that of the toilet bowl. Lift up existing toilet seat and place the seat directly onto the toilet bowl positioning the brackets towards the rear and the lip against the front of the bowl. Turn the adjustment knobs clockwise until a resistance is felt and the seat does not move from side to side. Ensure seat is firmly fixed before using.

**ATTENTION!** The side-locks have been especially designed to allow for easy tightening therefore turning should not be continued once a resistance is felt.

**Cleaning Instructions**  
All parts can be cleaned with warm water and a neutral cleanser. Alternatively, the seat can be totally immersed in water and/or can be sanified (without brackets) in an autoclave at 85°C for 1 minute.

**Cautions**  
Do not exceed maximum user weight: 200kg /415lb  
Always ensure the toilet seat is securely fixed before using. Do not hold onto seat for support while rising/lowering.

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